BG: ИНСТРУКЦИЯ ЗА УПОТРЕБА
EN: USER MANUAL
GR: ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
RO: INSTRUCȚIUNI DE UTILIZARE
ES: INSTRUCCIONES DE USO
DE: GEBRAUCHSANWEISUNG

РУ: ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ
IT: ISTRUZIONI PER L'USO
SR: UPUTSTVO ZA UPOTREBU
FR: NOTICE D'UTILISATION
HU: HASZNÁLATI ÚTMUTATÓ
NL: GEBRUIKERSHANDLEIDING

GENTLE TOUCH DUO

XN-D206F

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IMPORTANT! READ CAREFULLY AND KEEP FOR FUTURE REFERENCE!

THESE INSTRUCTIONS CONTAIN IMPORTANT INFORMATION, GUIDELINES AND RECOMMENDATIONS FOR THE PRODUCT AND ITS SAFE USAGE.

Congratulations on the purchase of this electric pump. This product is an electric breast pump with different levels of control of the suction power, unique valve with gentle suction and soft massage silicone gel pad. The Cangaroo electric breast pump provides many of the features which the busy mothers love and is perfect for many reasons:

- Expressing and collecting breast milk while you are away from your baby, for example when you are at work.
- Relieving the tension in your breasts caused by the large amount of breast milk. For example, when your baby's feeding habits change because it has started eating more solids.
- Maintaining your milk supply if you have to temporarily stop breastfeeding. For example, to express the unsuitable milk while you are undergoing medical treatment.

The Cangaroo electric breast pump has the following features:

- 1. Massage cycle, so called stimulation - 10 Levels and 3 frequency Levels.** This way the user can adjust the most suitable frequency of expression and stimulation, because the suction power range for massage is 16 kPa, with frequency from 70 to 90 times/minute.
- 2. Expression cycle - 15 Levels with 3 levels of frequency.** The user can adjust to the most suitable power and frequency of expression, because the suction power range of expression varies from 10-45 KPa.
3. Silent engine, ensuring discretion.
4. Light and easy to carry.
5. Easy to clean.

I. RECOMMENDATIONS AND WARNINGS FOR THE SAFE USAGE OF THE PRODUCT

READ CAREFULLY ALL INSTRUCTIONS BEFORE USING YOUR CANGAROO ELECTRIC BREAST PUMP!

1. Use this product only for its intended purpose as described in this manual
2. This unit is not intended for usage by persons (including children) with reduced physical, sensory or mental capabilities or the lack of experience and knowledge unless they have been given supervision or instruction by a person responsible for their safety.
3. **Warning!** The breast pump is intended for repeated use only by one user.
4. **Warning!** Children should not play with the manual breast pump.
5. **Warning!** Before first usage check the breast pump, including the adaptor for signs of damage. If you find such, do not use the product until the fault is removed. Do not try to modify or repair the product by yourself. For consultation contact an authorized service or the sales agent, from which you have bought the manual breast pump.
6. Do not use any accessories, which are not approved by the manufacturer and do not make any improvements or changes on the product as this may lead to its malfunction or injuries to the person, who uses it.
7. Do not use spare parts, which are not the original ones supplied by the manufacturer or the importer. Using such, Your warranty card will be invalid.
8. **WARNING: To reduce the risk of electrocution, please:**
 - Do not use while bathing or showering.
 - Do not place or store this product where it can fall or be pushed into a bathtub or sink.
 - Do not place or drop into water or other liquid.
 - Do not reach after a product that has fallen into water. Unplug from the wall socket immediately.
 - Never operate this product if it has a damaged adapter or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water.
 - Make sure the voltage of the AC adaptor is compatible with the power source.

- The product should not be left unattended when plugged into the electrical wall socket.
 - Do not approach high temperatures or fire sources during the use of the pump.
9. Do not use while you are pregnant as this may lead to stimulating the birth process.
 10. Never use while you are sleepy or drowsing.
 11. Do not leave it and do not keep the product in a place with too high room temperatures or at direct sunlight.
 12. Be careful not to drop the product and to put in its opening or in the feeding bottle other objects or liquids.
 13. If you are infected with Hepatitis A, Hepatitis B or HIV (Human Immunodeficiency Virus), pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.
 14. Do not attempt to remove the pump body from your breast while under vacuum. Turn the unit off and break the seal between your breast and the silicone pad with your finger, then remove the pump from your breast.
 15. Wash, rinse and sterilize all the parts of the breast pump after each use. Do not clean the part of the breast pump with strong detergents – alcohol-based ones, such containing bleach or with abrasive parts.
 16. Never wash and sterilize the electrical parts of the product – the motor of the pump and the AC adaptor in order not to cause permanent damage to the product.
 17. **WARNING!** This product is intended for personal usage, and it is not intended for the usage of other mothers.
 18. The breastmilk bottle in the set is an integral part of the breast pump. For the purposes of this product use only this bottle and only for the breast milk collected with this pump.

FOR YOUR CHILD'S SAFETY AND HEALTH

WARNING!

- Always use this product with adult supervision.
- Never use feeding teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Keep components not in use out of the reach of children.
- Throw away at the first signs of damage or weakness.
- Inspect carefully before each use. Pull the soother in all directions. Throw away at the first signs of damage or weakness.
- Do not leave the soother to direct sunlight or any other heat source, or in disinfectants (sterilizing solution) longer than recommended as this can weaken the teat.

II. PARTS

➤ **Figure PD:** 1. Dust shield; 2. Breast pad; 3. Cover; 4. Position; 5. Air tube; 6. Pump body; 7. Air hole; 8. Valve; 9. Pumping mechanism; 10. Baby bottle; 11. Adapter; 12. USB cable; 13. Connecting element; 14. Feeding teat with lid.

➤ **Figure DD – Display:** 1. Frequency; 2. Levels; 3. Symbol of mode; 4. Battery charge indicator; 5. Time; 6. Suction mode; 7. Massage mode.

➤ **Figure TD – Panel:** 1. ON/OFF; 2. Mode switch; 3. Frequency; 4. Level down; 5. Level up.

III. ELECTRICAL CHARACTERISTICS

- **Battery voltage:** AC100V – 240V; DC 5V, 1A
- **Battery capacity:** 1300 mAh

- **Charging time:** 180 min
- **Massage mode levels:** 10 levels
- **Suction mode levels:** 15 levels
- **Suction power range:** 10-45 kP
- **Baby bottle capacity:** 130 ml
- **Bottle material:** PP
- **Method of control:** buttons
- **Display:** LCD

IV. CHARGING THE ELECTRIC MOTOR OF THE PUMP

1. Before first time use charge the battery completely.
2. The breast pump is with a built-in battery 3.7V/ 1300 mAh.

3. Power connection:

- Plug the power adapter cable into the charging socket on the pump base.
 - Plug the power adapter into an electrical outlet.
 - Do not use the breast pump during charging, as its battery will be damaged.
4. Use only the charger provided with the breast pump. Do not use chargers from other products.
 5. Charge the battery on time. Charge as soon as the battery indicator is with 1 bar or the indicator starts flashing.
 6. During charging, the battery indicator will blink. Once the battery is fully charged, the indicator will stop flashing.
 7. If you will not use the product for a long time, charge the battery completely before storing it. Then charge the battery at least once a month to prolong its life.
 8. If the battery is damaged or cannot be charged anymore, do not try to open the product by yourselves. For consultation and repair contact the shop from where you bought the product or the importer/ distributor.
 9. The battery is recyclable. If the product is damaged and cannot be used any longer, take it to an official collection point for electronic products' waste, batteries, chargers. DO NOT throw the product away with the household waste. Follow your country's rules for the separate collection of electrical and electronic products and rechargeable batteries. Dispose of waste of this type only at the designated places. This will be important and significant contribution for the environmental protection and the reduction of the harmful effects of the substances contained in batteries and electronic components on the environment and human health.

V. ASSEMBLY OF THE ELECTRIC BREAST PUMP

ATTENTION! NEVER STERILIZE OR IMMEDIATE THE PUMP MECHANISM AND TUBE CONNECTING IT TO THE PUMP BODY - PART 5; See Figure AD.

- 1. Step 1:** Place the silicone valve (8) at the lower end of the pump body. Make sure that it is properly fixed around the ends of the pump body opening intended for it. Important! If the valve is loose or not installed correctly, it will not be possible to create a sufficient vacuum !.
 - 2. Step 2:** Fix the feeding bottle (10) to the pump body (6) by turning the bottle gently clockwise so that the thread of the bottle coincides exactly with the thread of the pump body
 - 3. Step 3:** Place the piston (4) in the upper part of the pump body. Make sure that it is properly fixed around the ends of the pump body opening intended for it.
 - 4. Step 4:** Put the protective cover (1) on the upper side of the pump body. Press and turn clockwise to fit in the correct position.
 - 5. Step 5:** With clean hands, insert the silicone nozzle (2) into the pump body (6). As you need to make sure it is placed tightly. Tip: If you are not going to use the breast pump, fit the protective cover (1).
 - 6. Step 6:** Attach the tubes (5) of the two pump bodies (6) to the connecting element (13). Then insert the tubes to the pump bodies. Attach the third end of the connecting element and the third tube. Then attach this tube to the pumping mechanism. So you can now use the breast pump.
- Follow the attached product assembly schedules to ensure proper assembly.

VI. USE

Using the pump - See Figure UD.

1. Wash your hands thoroughly and make sure your breasts are clean. Express some milk to check that your milk ducts are not blocked. The pump must be sterile and assembled.
2. Sit down and relax in a comfortable chair, leaning forward slightly (Use a soft and comfortable pillow as a backrest).
3. Put the pump on your breast and start the pump motor.

A. Massage (Stimulation):

This feature helps to stimulate the flow of breast milk. We recommend that you use this option first and then switch to squeezing mode. The massage regime mimics breastfeeding a baby, not too hard, and stimulates the flow of milk. The pumping mode also mimics baby sucking, but is stronger than the first mode, more even and with great efficiency. Press the mode selection button and activate the massage function. You can adjust the stimulation levels with the "+" and "-" buttons. They are a total of 10. And through the frequency button - the degrees of frequency. They are a total of 3.

B. Squeezing:

Press the mode dial and activate the squeeze function. With the "+" and "-" buttons you can adjust the drain levels. They are a total of 15. And through the frequency button - the degrees of frequency. They are a total of 3.

C. Memory:

The pump has a memory option - the last used function and level are automatically recorded before the pump is switched off. When you switch it on again, it will start working at the same function and level. **Note:** If during use and before switching off, the pumping mechanism has lost power due to low battery level, then the last used mode and level will not be memorized.

- If there is no pumping result, do not continue! Try using the pump at another time of the day.
- On average, you have to pump for 8 minutes to drain 120 ml of milk. Keep in mind that the amount of milk varies from woman to woman. If you find that you usually squeeze more than 120 ml per session, then you should use a larger feeding bottle.
- Do not overfill the milk bottle to prevent milk from leaking out of the bottle.
- When you stop using the pump, switch it off from the ON / OFF button. Then remove the tube from the back of the pump body. Remove the pump body from the bottle. Close the bottle with the spiral ring and the pacifier and place a lid on top to keep it sterile.

VII. WHEN TO USE YOUR BREAST PUMP

1. **If it is possible.** There should be a period of breastmilk secretion and breastfeeding before each use of the pump.
2. The electric pump is an apparatus, which helps postpartum women to ease the pain and to extend the breastfeeding time. It uses the premise that there is sufficient milk and smooth mammary gland. For some women, which are mothers for the first time, the mammary gland usually is not smooth enough. A reason sometimes can be the congenital nipple inversion or milk water shortage. So, we suggest you before purchasing and using the pump, to consult with a doctor who could confirm that your mammary gland is smooth and that the breast milk is in normal condition. Do not buy and use the pump only as a device to unblock your milk ducts.

VIII. HINTS

1. It takes practice in order to use the breast pump the best possible way. So it will take you at least several attempts before starting successfully to use the electric breast pump.
Choose a quiet time to use the pump – time, when no one will disturb you.
2. Sit in a comfortable chair, in a perpendicular position to the ground. Please, put a comfortable cushion behind your back before you start using the breast pump. Take a glass of water and place it near you, because you may feel thirsty during the pumping process.
3. Place a warm cloth on your breasts for a few minutes before starting to express the milk as this may facilitate the process and relieve the tension in the breasts.
4. Warmth and relaxation can help for the easier pumping of the breast milk, so before using the pump, you might like to take a bath or a shower.
5. Try expressing from one breast while your baby is feeding from the other, or continue expressing just after the feed.
6. Repositioning the pump on your breast from time to time can help stimulate the secretion. Practice in order to find the best pumping technique for you. If the process is causing you breast pain, stop immediately and consult with a doctor at once.
7. Make sure that you have not blocked or covered the air opening of the pump body.
8. Breast pump use errors:
9. The electric pump is an apparatus, which helps postpartum women to ease the pain and to extend the breastfeeding time. It uses the premise that there is sufficient milk and smooth mammary gland. For some women, which are mothers for the first time, the mammary gland usually is not smooth enough. A reason sometimes can be the congenital nipple inversion or milk water shortage. So, we suggest you before purchasing and using the pump, to consult with a doctor who could confirm that your mammary gland is smooth and that the breast milk is in normal condition. Do not buy and use the pump only as a device to unblock your milk ducts.

IX. HOW TO STORE THE BREAST MILK

1. Only breast milk that has been collected using a sterile breast pump can be stored to feed your baby.
2. The milk can be stored in a thermos at 40 ° C, so you can feed your baby every 4 hours.
3. If you store freshly expressed milk, seal the milk. The milk can be stored for 10 hours at room temperature at 19 - 25 ° C
4. The expressed milk can be stored in a refrigerator for a maximum of 48 hours (but not on the refrigerator door) or in a freezer for a maximum of 3 months.
5. Frozen milk must first be left in the refrigerator to thaw naturally. If you need it urgently, put it in a bowl of warm water and thaw it. After thawing, the milk can be stored in the refrigerator for 24 hours.
6. **NEVER FREEZE MILK AGAIN.**
7. Frozen milk can be gradually warmed in warm water or placed in a baby bottle warmer.
8. **NEVER WARM YOUR BREAST IN THE MICROWAVE, BECAUSE YOU CAN DESTROY VITAL NUTRIENTS AND ANTIBODIES THIS WAY.** Also, uneven heating can lead to too hot areas of breast milk, which can cause burns to your baby's upper digestive tract. Never place a frozen bottle of milk in boiling water, as this may cause the bottle to burst / break. Never pour boiling water directly into a feeding bottle. You should allow the water to cool for about 20 minutes before filling the bottle.

X. CLEANING AND STERILIZATION OF THE PUMP

Important: NEVER CLEAN WITH WATER or STERILIZE the LCD display of the pump and the connecting hose (5). You can only clean it with a soft dry cloth.

Clean and sterilize your electric pump before first use and after each use, following the instructions below.

❖ Sterilization:

1. Remove the tube (5) from the pump mechanism (9) and set it aside. Separate all other parts, 1, 2, 3, 4, 6, 7, 8, 10, 13, 14 then wash them in warm soapy water and dry them. Do not use antibacterial or corrosive cleaners.
2. Sterilize ONLY the following parts 1, 2, 3, 4, 6, 8, 10 and 14 in a steam sterilizer or by boiling for 10 minutes.

❖ RECOMMENDATIONS FOR CLEANING

1. Use a steam sterilizer. Boiling the parts is not recommended. The first reason is that the product will be easily damaged if improper sterilization is applied. Second, this method of sterilization cannot achieve full effect.
2. If you do decide to use boiling as a sterilization method, pour more water into the container so that the parts are completely covered with water. You must not leave the product unattended during the sterilization process. Do not boil for too long - maximum 10 minutes.
3. For breastfeeding mothers who breastfeed every 2-3 hours, we do not recommend sterilizing and cleaning after each use. Excessive disinfection and cleaning can easily lead to premature aging of parts and shorten the life of the breast pump. Thorough cleaning of stains and sterilization once a day is recommended.
4. Store the parts in a clean container until the next use.

IMPORTANT! If you use the breast pump as a means of storing milk, you must take care to disinfect all washable parts of the pump, because otherwise, the milk can easily spoil, as it is a product that is difficult to store.

ATTENTION! Before use, make sure that the pump body, valve, piston and cover are dry.

•DO NOT STERILIZE THE TUBES – PART 5 (3 pcs), to avoid accidental ingress of water into the pump motor.

•If you use a steam sterilizer to clean the pump parts, be careful that they do not come into contact with the heating plate of the sterilizer.

XI. TROUBLESHOOTING GUIDE

1. Lack of suction:

- Check if you have assembled all parts correctly and if the valve is positioned correctly.
- Ensure that the silicone massage pad and the shield brace are fitted securely to the pump body and a perfect seal is created.
- Check if you have placed the silicone valve firmly to the opening of the pump body.
- Check whether there are any damaged parts.

If you can not turn the pump on, check whether the battery is exhausted. Connect the adapter to the terminal for charging the motor.

2. No milk is being expressed:

- Check point 1
 - Ensure the pump is correctly assembled and that suction is being created.
 - Relax and try again.
 - There are 15 levels of expression, the maximum power is 42kPa; the average suction power of the baby is 20 – 30 kPa. We recommend you increase the levels gradually; the most suitable level is the one, with which the suction is painless.
- The following is NOT true: the stronger – the better.** Too strong suction can hurt the nipple.

3. Pain in the breast area where there is expression:

- If you feel pain and no milk flows during expression, maybe your mammary gland is blocked. This case may result in mastitis. Contact a physician promptly. The complete milk expression will have good effect on mastitis protection.
- You may be using too high level of suction. You do not need to use all the suction that the pump can generate.
- Try to adjust the proper suction and frequency.
- Consult your health professional/ breastfeeding advisor.

4. Flow is too slow:

The speed of milk flow differs for every woman. The higher degree and level of expression do not mean faster flow of milk. This depends on many factors – constitution of the breast, level of possibility, even the mood.

- The more regularly you use the breast pump, the quicker and easier the flow of milk. Before you start using it, put a warm cloth on your breasts and use the massage mode for about 5 minutes before you switch to expression mode.

5. Milk is dropping down the silicone massage pad:

- Remove and refit again the massage pad in order to ensure it is firmly attached to the central part of the funnel.
- Try to lean slightly forward.

6. Cracking and discoloration of the pump/ feeding bottle parts:

Avoid contact with antibacterial or abrasive detergents/ cleaners as they can damage the plastic. Combination of detergents, cleaning products, sterilizing solution, softened water and temperature fluctuations can cause plastic to crack. If this occurs, please, do not use. The pump body and the breast milk containers are dishwasher safe but the food colorings may cause discoloring them. Keep the product out of direct sunlight as prolonged exposure may lead to discoloration.

7. Lost or broken parts:

For spare parts contact an authorized service center or the sales agent from whom you have purchased the product. Do not use spare parts, which are not approved or supplied by the manufacturer.

- Do not attempt to repair or to modify the product, because this may endanger Your and Your baby's health.

Thank you for purchasing this product! Use it carefully and follow the instructions from this manual in order to ensure the safe use and maintenance of the electric breast pump. When having questions regarding the use, maintenance and repair of the product, please, contact the importer or sales agent from whom you have purchased the product.

Thank you for purchasing this product! Use it carefully and follow the instructions from this manual in order to ensure the safe use and maintenance of the electric breast pump. When having questions regarding the use, maintenance and repair of the product, please, contact the importer or sales agent from whom you have purchased the product.

XII. CONFORMITY WITH EUROPEAN STANDARDS AND DIRECTIVES

This product has been designed and manufactured in accordance with the requirements of the European Directives: the Electromagnetic Coherence Directive 2014/30/EU; Directive 2011/65/EU and its supplements 2015/863/EU and 2017/2102/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment, Directive 2011/8/EU on the restriction of the use of Bisphenol-A in plastic infant feeding bottles; Regulation 10/2011/EU on plastic materials and articles intended to come into contact with food. **The product complies with the requirements of the following European standards, some of which are harmonised:** EN 55014-1:2017; EN 55014-2:2015; EN 61000-3-2:2014; EN 61000-3-3:2013; EN 61000-4-2:2008; EN 61000-4-3:2010; EN 61000-4-4:2012; EN 61000-4-5:2005; EN 61000-4-6:2013; EN 61000-4-11:2004.

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Gemaakt voor CANGAROO BELANGRIJK: Moni Trade Ltd.